Air Pressure Massage System

Model Nymph 4

Nymph 6

Nymph 8

NL – Nederlands .4

1. INLEIDING 5

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3. TERMINOLOGIE 8

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4.1. Contra-indicaties 9

4.2. Algemene veiligheid 9

4.3. Gebruikers 10

4.4. Elektrische veiligheid 11

4.5. Netsnoer 12

4.6. Warmte & ventilatie 13

4.7. Water 14

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Air Pressure Massage System

Model Nymph 4-6-8

This user manual accompanies the Nymph 4-6-8 and contains important information about installation, safety, disposal and use. Read this manual carefully and follow the instructions to ensure safe and proper use of the product.

Keep the manual for further reference, even if you are familiar with the product. Make sure to include this user manual when passing on this product to third parties.

The images in the user manual are for illustrative purposes only and may differ slightly from the product. In that case, **NYMPH** is not obliged to update or revise the user manual. Use the actual product as a reference.

IMPORTANT – READ BEFORE USE

This device is a Class IIa medical device intended for intermittent pneumatic compression therapy.

Use this device only in accordance with its intended use and observe all contraindications, warnings and precautions described in this user manual.

NYMPH accepts no liability for damage, injury or loss resulting from use contrary to this user manual, failure to observe contraindications, warnings or precautions, or use outside the intended purpose.

1. INTRODUCTION

Thank you for choosing a **NYMPH** product. This manual provides the information required for safe and proper operation of the product.

This product bears the CE marking and complies with applicable European regulatory requirements.

	CE marking This product bears the CE marking, indicating that it conforms to the relevant European Union directives concerning safety, health, and environmental protection. The CE marking is a declaration by the manufacturer that the product meets the applicable EU legal requirements.
	RoHS Compliance This product complies with the RoHS Directive (2011/65/EU), which restricts the use of certain hazardous substances in electrical and electronic equipment. Compliance ensures that levels of substances such as lead, mercury, and cadmium remain below the maximum permitted thresholds.
EMC	EMC Compliance This product complies with the Electromagnetic Compatibility (EMC) Directive (2014/30/EU), which ensures that the device functions properly in its environment without causing or suffering from electromagnetic interference.
MDR	MDR Compliance This product complies with the Medical Device Regulation (EU) 2017/745 (MDR), which establishes requirements for the safety, performance, and clinical evaluation of medical devices placed on the European market. Compliance demonstrates that the product meets the essential requirements for medical devices under EU law.

The full EU Declaration of Conformity and all supporting test documentation are available upon request.

If you have any questions, remarks, or problems using the product, please contact us.

We hope you enjoy your purchase!

2. EXPLANATION OF SYMBOLS

Throughout this user manual, the following symbols are used:

	NOTE: Provides important information or useful tips for the use of the product.
	DANGER: Indicates a hazardous situation which, if not avoided, will result in death or serious injury.
	WARNING: Indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	CAUTION: Indicates a hazardous situation which, if not avoided, could result in minor injury or material damage.

The safety symbols do not replace the written safety instructions. Read and follow the safety instructions exactly.

Medical device symbols:

Symbol	Symbol title	Explanation
	Serial number (SN)	Indicates the manufacturer's serial number for product identification.
	Batch code (LOT)	Indicates the manufacturer's batch or lot number.

	Authorised representative in the European Union (EU REP)	Indicates the authorised representative within the European Union.
	Date of manufacture	Indicates the date when the medical device was manufactured
	CE marking with notified body number	Indicates conformity with applicable EU legislation. The number identifies the notified body.
	Type BF applied part	Indicates a Type BF applied part providing protection against electric shock.
	WEEE symbol	Indicates that the device must not be disposed of with household waste. Dispose of the product in accordance with local regulations for waste electrical and electronic equipment (WEEE).
	Indoor use only	Indicates that the device is intended for indoor use only.
	Class II equipment	Indicates Class II equipment with double or reinforced insulation for protection against electric shock.
	IP21 protection rating	Indicates protection against solid foreign objects larger than 12,5 mm and protection against vertically falling water droplets.
	Manufacturer	Indicates the legal manufacturer of the product.

3. TERMINOLOGY

NOTE:

In this user manual, the term “Nymph 4-6-8” collectively refers to the Nymph 4, Nymph 6 and Nymph 8 models. Unless otherwise stated, the term refers to the complete product, including the control unit (device) and the supplied accessories such as the remote control and cuff with connecting hose. When individual components are described, they are referred to by their specific names.

4. SAFETY INSTRUCTIONS

Follow the instructions and safety information in this user manual carefully. Failure to do so may result in personal injury or damage to property.

Damage or injury resulting from improper use or failure to follow these instructions is not covered by the warranty.

Electrical devices and batteries may present hazards if used incorrectly. These may include injury, fire, electric shock, or damage to property. Always handle the device with care and keep it out of the reach of children.

This user manual provides the information required for safe use of the device. However, not all possible situations or hazards can be covered. Always use the device with caution and follow the instructions provided.

Residual risks may remain despite all safety measures. Follow all instructions and warnings in this user manual to reduce the risk of injury, damage, or malfunction.

If you have any questions that are not addressed in this user manual, please contact us.

4.1. Contraindications

Do not use the device:

- on open wounds, fractures, or severely injured body areas;
- on areas affected by acute inflammation or infection;
- if you have acute venous thrombosis or deep thrombophlebitis;
- if you suffer from pulmonary edema;
- if you have severe peripheral arterial occlusive disease (PAOD).

Consult a physician before use if you:

- are pregnant;
- have a pacemaker or other implanted electronic device;
- suffer from heart disease, circulatory disorders, or uncontrolled hypertension;
- have diabetes with reduced sensation or peripheral neuropathy;
- are undergoing cancer treatment;
- take anticoagulant medication;
- have any other medical condition that may be affected by compression massage.

4.2. General safety

- Do not use the Nymph 4-6-8 for any purpose other than its intended use (see the chapter "INTENDED USE").
- The Nymph 4-6-8 is designed and intended for household use and is not suitable for commercial or industrial use.
- Do not use the device in places where the use of electronic equipment is not allowed.
- The Nymph 4-6-8 is intended for indoor use only. Do not use the device outdoors.
- Handle the device carefully. Shocks, impacts, or falls, even from a low height, can damage the device.
- Do not place the Nymph 4-6-8 on an unstable surface. This may cause it to fall and result in a malfunction.
- Do not squeeze the cuff.
- Do not expose the Nymph 4-6-8 to excessive vibration, crushing forces, impacts, or other mechanical stress.
- Do not sit on the cuff.

- The device must be properly and securely connected. Follow the instructions to prevent any possible damage or injury.
- Use only the accessories or attachments supplied or recommended by the manufacturer for this device. Incorrect accessories or attachments may pose a risk to the user and can damage the Nymph 4-6-8.
- Do not insert any objects or fingers into openings.
- Do not apply excessive force when using the buttons on the Nymph 4-6-8.
- Do not touch the cuff with sharp objects.
- Do not apply excessive force to the display, and do not touch the display with sharp objects.
- Never place objects on the Nymph 4-6-8.
- Do not leave the switched-on Nymph 4-6-8 unattended. Close supervision is necessary when using near young children.
- Do not use the Nymph 4-6-8 near strong magnetic fields, e.g. motors, transformers, mobile phones, etc.
- Do not use the Nymph 4-6-8 in dusty environments.
- If it is no longer possible to operate the Nymph 4-6-8 safely, take it out of operation and protect it from any accidental use. A safe operation can no longer be guaranteed if the device:
 - is visibly damaged;
 - is no longer working properly;
 - has been stored for extended periods in poor ambient conditions;
 - or has been subjected to serious transport-related stresses.

4.3. Users

- The Nymph 4-6-8 is not a toy. Keep it out of reach of children and pets.
- The device may only be used by adults. The cuffs are designed for adult users only. Using them on children may result in injury.
- Do not allow persons who are not familiar with the operation of the Nymph 4-6-8 or with these instructions to use the device.
- The device may only be used by persons who understand these instructions and are aware of the potential hazards.
- This device is not intended for use by persons with reduced physical, sensory, or mental capabilities or a lack of experience and knowledge, unless they have been given supervision or instruction concerning the use of the device by a person responsible for their safety.
- Cleaning and maintenance must not be performed by children.

- Do not operate the Nymph 4-6-8 when you are tired, distracted, or under the influence of alcohol or medication.

4.4. Electrical safety

- You must not modify, repair yourself, or disassemble the Nymph 4-6-8. This can damage the device.
- Maintenance, modifications, and repairs may only be carried out by the manufacturer, a professional workshop appointed by the manufacturer, or a similarly qualified person.
- Check the device from time to time for damage. Never use the device if it shows signs of damage.
- Do not use any other mains plug for this device. This can cause failure and/or damage.
- Ensure your mains outlet supplies the correct power voltage to avoid damage to the device.
- The mains outlet must be located near the Nymph 4-6-8 and be easily accessible.
- To reduce energy consumption and potential safety risks, always unplug the device when it is not in use, during cleaning or maintenance, or if a fault occurs. The device remains connected to the mains as long as it is plugged in.
- Do not turn the Nymph 4-6-8 on and off by plugging and unplugging it. Always use the ON/OFF button.
- Never force the mains plug of the power cord into a mains outlet. If it does not fit easily into the mains outlet, stop using it. Always ensure that the mains plug of your device is properly connected to the mains outlet.
- If the power fails during use, turn off the device immediately and pull the mains plug from the mains outlet.
- If smoke is produced, or there are any unusual sounds or smells, switch off the Nymph 4-6-8 immediately and remove the mains plug from the mains outlet. In these cases, the Nymph 4-6-8 should not be used until it has been inspected by the manufacturer or a similarly qualified person. Never inhale smoke from a potential device fire. If you have inhaled smoke, however, consult a doctor. The inhalation of smoke can be harmful.
- In the event of a fire, disconnect the power supply if this can be done safely. Use suitable fire-fighting equipment and keep a safe distance from the device.
- For safety reasons, always unplug the device in case of a thunderstorm.

4.5. Power cord

- Fully unwind the power cord before use.
- Do not modify the power cord.
- Check the power cord periodically for damage. Never use the power cord if there are any signs of damage.
- To prevent risks, damaged power cords may only be replaced by the manufacturer, a professional workshop appointed by the manufacturer, or a similarly qualified person.
- Do not touch the power cord if it is damaged. A damaged power cord can cause a deadly electric shock. First, power down the respective mains outlet (e.g. via the respective circuit breaker) and then carefully pull the mains plug from the mains outlet. Never use the device if the power cord is damaged.
- Never pull the mains plug from the mains outlet by pulling at the power cord. Always pull it from the mains outlet using the intended grips.
- Do not carry or pull the Nymph 4-6-8 by its power cord.
- Never wrap the power cord around the device. This will cause the cord to wear prematurely and break.
- Ensure that you do not trip over the power cord.
- Do not let the power cord hang over the edge of a table or counter.
- The power cord should not be kinked, trapped, or twisted.
- Ensure that the power cord is not squeezed, bent, damaged by sharp edges, or put under mechanical stress.
- Never tie a knot in the power cord.
- Untwist the power cord periodically if it becomes twisted due to use.
- Avoid excessive thermal stress on the power cord from extreme heat or cold.
- If using an extension cord attached to the mains outlet, ensure that the total electric power consumption of connected devices does not exceed the rated capacity.
- Children can become entangled while playing with cords, posing a strangulation hazard. Cords are not toys.

4.6. Heat & ventilation

- The Nymph 4-6-8 may become warm during operation. This is normal.
- Do not cover the device or the ventilation holes while it is in operation, as this may cause the Nymph 4-6-8 to overheat. This poses a risk of fire, electric shock, or injury.
- Ensure that the ventilation around the Nymph 4-6-8 is good during use.
- Keep the device away from heat sources and away from areas where the ambient temperature is relatively high (e.g. stoves or other heat sources). Never use the device near naked flames (such as burning candles) or on hot surfaces.
- The Nymph 4-6-8 and remote control are designed to work best in normal ambient temperatures between 0 °C and 35 °C and to be stored at temperatures between 0 °C and 45 °C. The Nymph 4-6-8 and the remote control may be damaged, and their lifespan may be shortened if the devices are used or stored outside this temperature range. Do not subject your devices to dramatic changes in temperature. Do not store your devices in a hot car or direct sunlight for extended periods. Do not place the devices near a window (especially in the summer).

4.7. Water

- This device is not waterproof. Exposure to water may result in electric shock, short circuits, or permanent damage.
- Never use the Nymph 4-6-8 with moist hands. Moisture can cause electric shock or short circuits.
- Never touch the plug with moist hands while connecting or disconnecting it from the mains outlet. Moisture can cause electric shock or short circuits.
- Do not use the Nymph 4-6-8 in or near a bath, shower, swimming pool, or any container filled with water, such as a sink.
- Do not use in wet environments. Do not immerse the device, power cord, plug, cuff or remote control in any liquid. Never pour liquids over or near the Nymph 4-6-8. There is a risk of electric shock or a short circuit. If liquid does get into the housing, unplug it immediately and then turn off the Nymph 4-6-8.
- Do not reach for the device if it has fallen into the water. Immediately unplug it from the mains outlet and only then remove it from the water.
- Ensure that the Nymph 4-6-8 is placed on a dry surface.

4.8. Battery safety (remote control)

- Do not place the remote control near a fire or in direct sunlight. The heating of the device can cause additional heating, breaking, or explosion of the battery.
- Do not attempt to pierce, modify or disassemble the battery, as this may result in fire, explosion, chemical leakage, or serious injury.
- Do not use a battery that is damaged or deformed.
- Do not use your remote control if the battery begins to emit odour, starts to overheat, or begins to leak.
- Do not touch any leaking materials or breathe fumes emitted from the battery. Leaking or damaged batteries can lead to caustic burning of the skin. Therefore, use suitable gloves.
- The battery contains hazardous substances. Do not open the battery or insert any objects into it.
- Never throw the battery into a fire.
- When you handle the battery, ensure your hands, the product, and the battery are dry.
- Do not short-circuit. Short-circuiting may cause the battery to explode or leak.
- To avoid accidental short-circuiting of the battery after removal, do not let the battery terminals come into contact with metal objects (e.g. coins, hairpins, rings). Do not wrap the battery in aluminium foil. Tape the battery terminals or put batteries separately in a plastic bag before you discard them.
- Be careful not to drop or bump the battery.
- In case of battery ingestion: immediately seek medical advice or contact your local poison control centre for treatment advice.
- Clean the battery contacts and also those of the device before battery installation.
- Ensure the battery is installed correctly with regard to polarity (+ and -).
- Examine your remote control and ensure the battery compartment is correctly secured, e.g. that the screw or other mechanical fastener is tightened. If the battery compartment does not close securely, stop using the remote control and keep it away from children.
- Only replace the battery with the same size and the same type of battery.
- Use the battery within the recommended usage period.
- Prevent children or animals from coming into contact with the battery.

4.9. Residual Risks

Despite the implementation of all safety measures, certain residual risks may remain when using the Nymph 4-6-8.

- Temporary discomfort, skin irritation, redness, numbness, or pressure marks caused by prolonged use or excessive pressure settings.
- Discomfort or injury resulting from improper placement or incorrect fitting of the cuffs.
- Reduced blood circulation, pain, or physical discomfort caused by excessive treatment duration or unsuitable pressure settings.
- Electric shock, overheating, or fire hazards resulting from damaged cables, damaged components, improper repairs, exposure to liquids, or misuse of the device.
- Tripping hazards caused by hoses, cables, or connected garments during use or storage.
- Damage to the cuffs, hoses, connectors, or control unit caused by sharp objects, excessive bending, pulling, twisting, crushing, or improper storage.
- Risks associated with use by persons with contraindicated medical conditions, including circulatory disorders, thrombosis, severe hypertension, implanted medical devices, or other conditions listed in this manual.
- Risks associated with use by children, persons with reduced physical, sensory, or mental capabilities, or persons unable to understand the instructions or communicate discomfort.

To minimise these risks, always follow all instructions and safety warnings provided in this manual.

4.10. Reasonably foreseeable misuse

The following uses are considered reasonably foreseeable misuse and must be avoided:

- Using the device for purposes other than intermittent pneumatic compression therapy using the supplied compression garments.
- Using the device on infants, children, pets, or persons unable to safely operate the device or communicate discomfort.
- Using the device while sleeping, unconscious, under the influence of alcohol, medication, or substances that may reduce awareness or sensitivity.
- Using the device with damaged cuffs, hoses, connectors, cables, plugs, or other components.
- Using the device in wet environments, near water, or with wet hands.
- Covering the control unit during operation or blocking the ventilation openings.
- Pulling, twisting, sharply bending, crushing, or stepping on hoses or power cables.
- Disconnecting the cuffs or hoses while the device is operating or pressurised.
- Using pressure settings, treatment durations, or programs that cause pain, numbness, or significant discomfort.
- Using the device despite known contraindications, medical conditions, injuries, or after medical advice not to use compression therapy devices.
- Attempting to open, modify, repair, or disassemble the device or accessories.
- Inserting foreign objects into the control unit, connectors, ventilation openings, or other openings of the device.
- Using non-approved accessories, replacement parts, power cables, or attachments.
- Standing, walking, or moving around while connected to the compression garments.
- Using the product outdoors or exposing it to direct sunlight, excessive heat, open flames, or freezing temperatures.

Such misuse may result in injury, fire hazards or damage to the device.

4.11. Medical considerations

This product is a Class IIa medical device intended for intermittent pneumatic compression therapy. It should only be used in accordance with its intended purpose and should not replace professional medical advice, diagnosis, monitoring, or treatment.

Consult a physician before using this device if you:

- suffer from circulatory disorders, vascular disease, varicose veins, diabetes, severe hypertension, or heart disease;
- have implanted medical devices such as a pacemaker;
- are pregnant or recently underwent surgery;
- experience chronic pain, numbness, reduced sensitivity, or skin conditions;
- are undergoing medical treatment or have any other medical concerns.

Do not use the device on injured, swollen, infected, inflamed, or sensitive body areas unless approved by a qualified healthcare professional.

Stop using the device immediately and seek medical advice if you experience pain, dizziness, numbness, skin discolouration, unusual swelling, breathing difficulties, or any other abnormal physical reaction during or after use.

Users are responsible for determining whether this product is suitable for their individual physical condition.

5. INTENDED USE

The Nymph 4-6-8 is intended for intermittent pneumatic compression therapy using inflatable compression garments for the legs, arms, waist or lower body. The device provides sequential air pressure compression.

The device is intended to support blood circulation and temporarily relieve minor muscle discomfort and fatigue.

**WARNING:**

Use this product only as described in this manual. Use outside the intended purpose or contrary to the instructions may result in reduced performance, damage to the device, or safety risks.

6. SCOPE OF DELIVERY

6.1. Contents

The following components are included in the scope of delivery:

- 1x Nymph 4-6-8
- 1x Remote control (without battery)
- 1x Air hose set
- 1x Cuff
- 1x Carrying bag
- 1x Shoulder strap for carrying bag
- 1x User manual

**NOTE:**

Due to transport and packaging requirements, some accessories may be supplied disassembled. Follow the assembly instructions carefully.

6.2. Unpacking

Remove everything from the packaging and check for damage. If any damage is detected, do not use the Nymph 4-6-8. Be very careful when opening the package to avoid damage to the product.

After unpacking the product, verify that the delivery is complete and that all parts are in good condition. If a part is missing or damaged, please contact us.

Before use, remove all packaging materials.

NOTE:

This device is packaged to protect it during transport. Keep the original box and other packaging in a safe place to prevent damage to the device if you need to transport it in the future. If you do throw away the packaging, please ensure it is recycled properly.

6.3. Safety during unpacking

Do not leave packaging materials and accessories lying around carelessly during unpacking. They may become a dangerous plaything for children.

WARNING: Suffocation hazard

Keep plastic bags and foil out of the reach of babies and children to prevent suffocation. Thin foils can stick to the nose and mouth and obstruct breathing.

WARNING: Strangulation hazard

Cords and hoses longer than 22 cm can be harmful to babies and very young children.

Children may become entangled while playing with the power cord and hoses. Do not leave the accessories lying around during unpacking.

WARNING: Coin/button cell battery.

The remote control is delivered without the necessary button cell battery. Be careful when unpacking and/or installing batteries. Button cell batteries are small and very powerful. To kids, they might look like candy or coins, and can be swallowed very easily.

- Keep new and used button cell batteries out of reach of children.
- A swallowed button cell battery can cause internal chemical burns in as little as two hours and lead to death.
- If you think a battery might have been swallowed or placed inside any part of the body, seek immediate medical attention.
- Dispose of used button cell batteries immediately.

7. TECHNICAL DATA

NOTE:

Due to continuous improvements to our product, the technical specifications may be changed without prior notice.

Product	Air pressure massage system		
Model	Nymph 4	Nymph 6	Nymph 8
Brand	Nymph		
Colour	Black		
Material	Hard plastic		
Number of air chambers	4	6	8
Number of programs	6 (A – F)		
Power	65 W		
Input voltage	220 – 240 V		
Frequency	50/60 Hz		
Protection class (device)	Class I 		
Fuse	T3.15A, 5 × 20 mm		
Power cable length	1.5 m		
Timer setting	1 – 90 min		
Pump output	20 – 25 l/min		
Pressure	30 – 240 mmHg (30 mmHg increments)		
Operating temperature	10 – 40 °C		
Storage temperature	10 – 40 °C		
Operating humidity	30 – 85 % RH		
Storage humidity	30 – 85 % RH		
Remote control power supply	CR2032 (button cell battery)		

IP rating	IP 21
Weight	2.3 kg
Dimensions	300 × 237 × 126 mm

Product identification:

Brand	Nymph		
Model	Nymph 4	Nymph 6	Nymph 8
OEM Manufacturer	Xiamen Weiyou Intelligent Technology Co., Ltd., Xiamen, China		
OEM Model	VU-NIPC02	VU-IPC04	VU-IPC06

NOTE:

The product identification and rating label is located on the device. This label contains important information such as the model number and electrical ratings.

MDR Compliance:

This product is a Class IIa medical device and complies with the Medical Device Regulation (EU) 2017/745 (MDR), which establishes requirements for the safety, performance, and clinical evaluation of medical devices placed on the European market. Compliance demonstrates that the product meets the essential requirements for medical devices under EU law.

Notified Body:

Notified Body 0598 – SGS Fimko Oy
 Takomotie 8, FI-00380 Helsinki, Finland
 Tel.: +358 9 696 361
www.sgs.fi
 Business ID 0634247-4

Authorised representative in the European Union (EU REP):

EU


REP

SUNGO Cert GmbH

Harffstr. 47, 40591 Düsseldorf, Germany

8. PRODUCT OVERVIEW

8.1. The control unit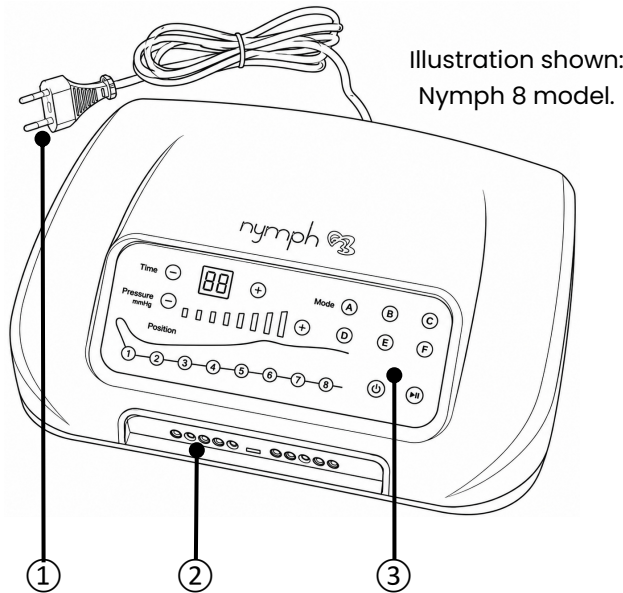

- ① Power cord
- ② Multi-port connector socket
- ③ Operating panel

8.2. The operating panel

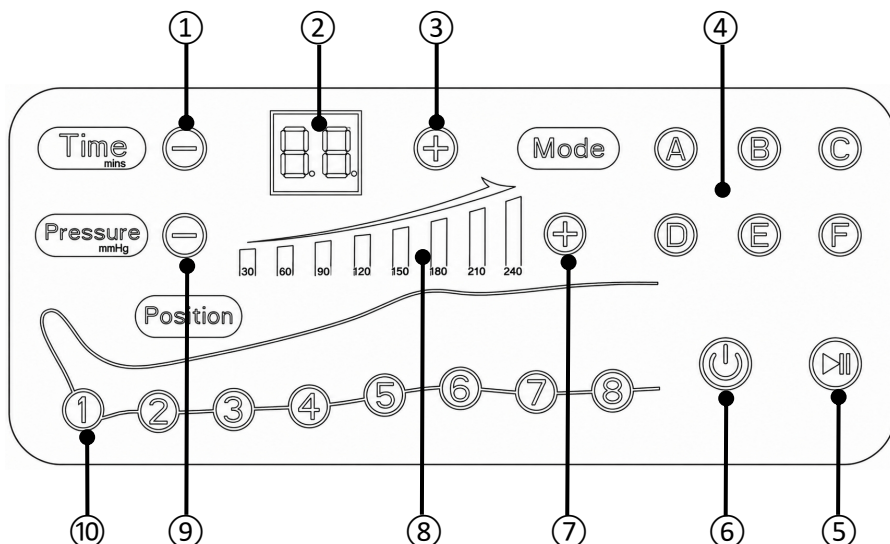
Illustration shown: Nymph 8 model.

- ① Subtract time button
- ② Remaining time indicator (1 - 90 minutes)
- ③ Add time button
- ④ Program mode buttons (A, B, C, D, E, F)
- ⑤ Start/Pause button
- ⑥ ON/OFF button
- ⑦ Add pressure button
- ⑧ Pressure indicator (8 levels: 30, 60, 90, 120, 150, 180, 210, 240 mmHg)
- ⑨ Subtract pressure button
- ⑩ Position/Chamber buttons
 - Nymph 4: 1, 2, 3, 4
 - Nymph 6: 1, 2, 3, 4, 5, 6
 - Nymph 8: 1, 2, 3, 4, 5, 6, 7, 8

8.3. The remote control

- ① Subtract time button
- ② Add time button
- ③ Subtract pressure button
- ④ Add pressure button
- ⑤ Position/Chamber buttons
 - Nymph 4: 1, 2, 3, 4
 - Nymph 6: 1, 2, 3, 4, 5, 6
 - Nymph 8: 1, 2, 3, 4, 5, 6, 7, 8
- ⑥ Mode button
- ⑦ Start/Pause button


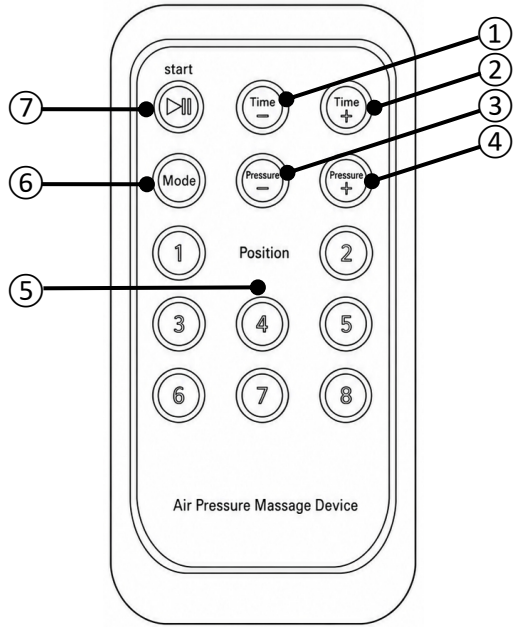
Illustration shown: Nymph 8 model.

9. MESSAGE PROGRAMS A–F

The air chambers inside the leg cuffs inflate and deflate according to the selected program. This creates a sequential compression massage designed to promote comfort and relaxation. The treatment time, pressure level, and additional settings can be adjusted at any time during the session.

- A** Normal
- B** Sequential squeeze
- C** Double wave
- D** Whole squeeze
- E** Sequential squeeze + double wave
- F** Normal + double wave + whole squeeze

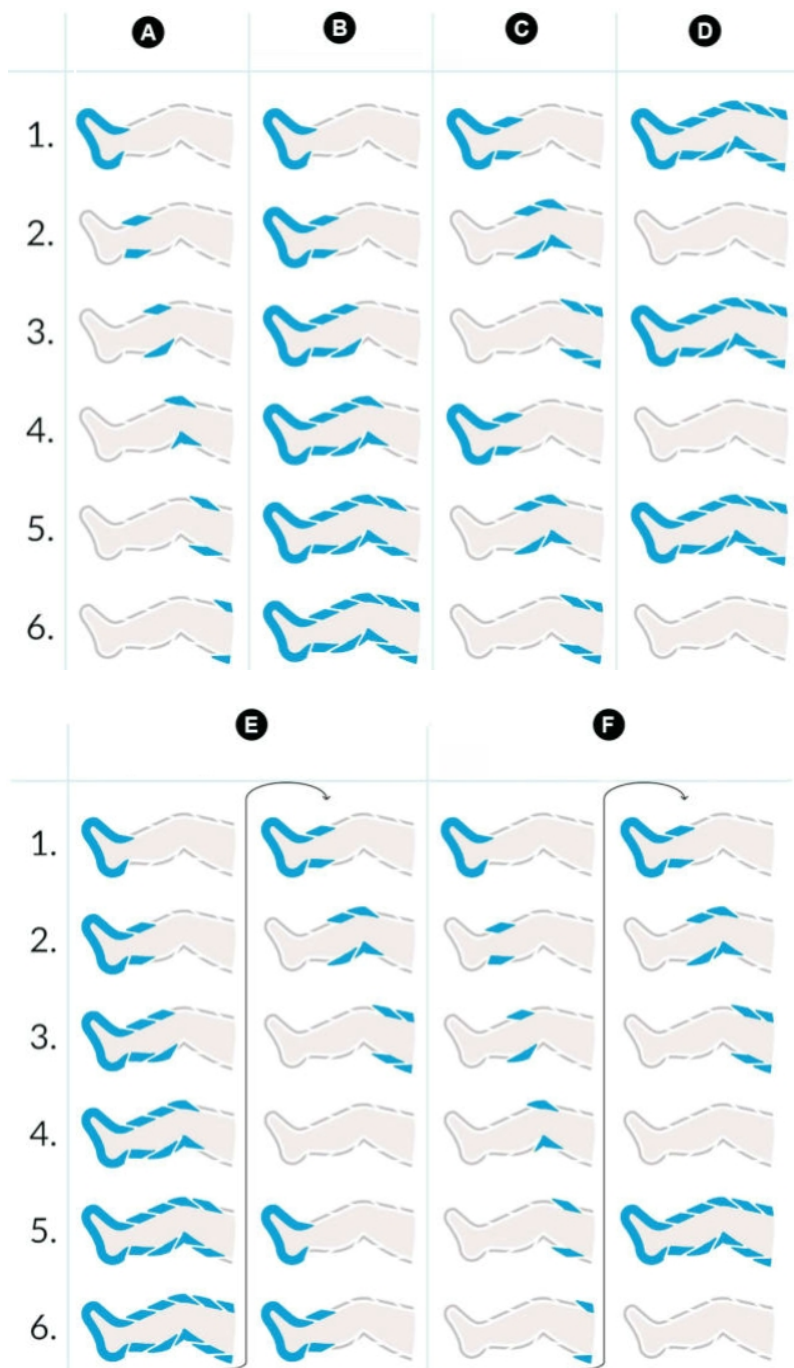10. BENEFITS OF COMPRESSION MASSAGE

The device provides a sequential compression massage using multiple air chambers that inflate and deflate according to the selected program. This creates a gentle pressure massage designed to enhance comfort and relaxation.

Potential benefits of regular use may include:

- Relief from the feeling of tired or heavy legs after prolonged standing, sitting, or daily activities.
- Increased comfort and relaxation after physical activity or exercise.
- A soothing massage experience for the legs or arms.
- Temporary relief from muscle tension and physical fatigue.
- Improved comfort during periods of rest and recovery.

NOTES:

- The benefits described above are intended to support compression therapy and individual experiences may vary.
- This medical device is intended for intermittent pneumatic compression therapy as described in this manual. It is not intended to replace professional medical advice, diagnosis, or treatment.


11. OPERATION

11.1. Before first use

FIRST USE:

Open the filter cover and remove the transportation screw by turning it counterclockwise. The screw protects the control unit during transport and must not be reinstalled.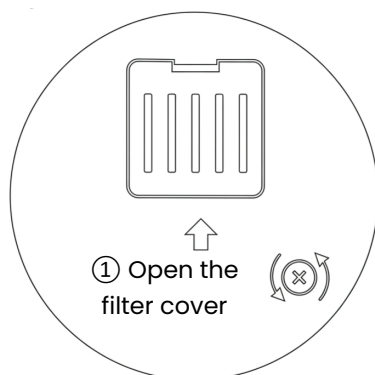


② Remove the transportation screw by turning it counterclockwise

If the remote control contains a battery insulation strip, remove it before first use. If the battery is not installed (for example, due to transport regulations), insert a CR2032 battery into the battery compartment.

11.2. Connecting the system


Step 1

Connect the power cord to an earthed 220–240 V mains outlet.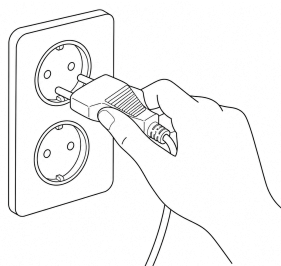

Step 2

Check whether the air hose is correctly connected to the cuff(s). If it is already connected, proceed to Step 3. If not, insert the hose pins into the connectors on the cuff(s).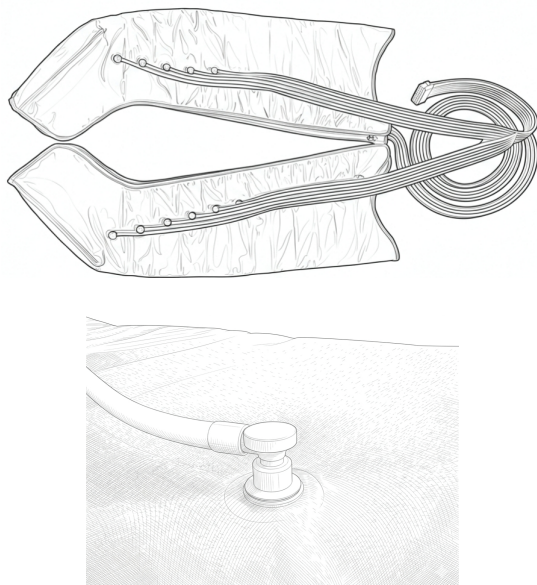

Step 3

Connect the main connector of the air hose to the control unit.

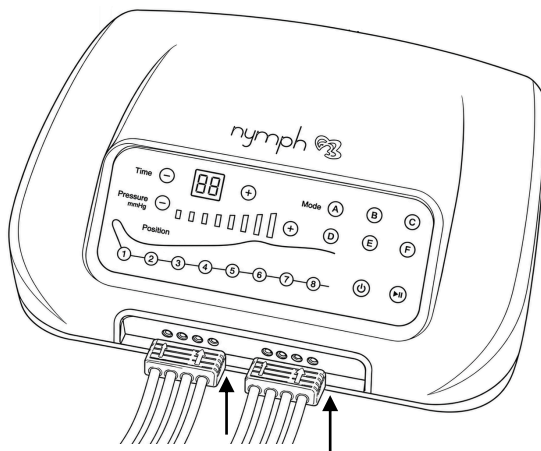
Illustration shown: Nymph 8 model.

Step 4

Check that all connections are secure and airtight.

Step 5

Put on the cuff(s).

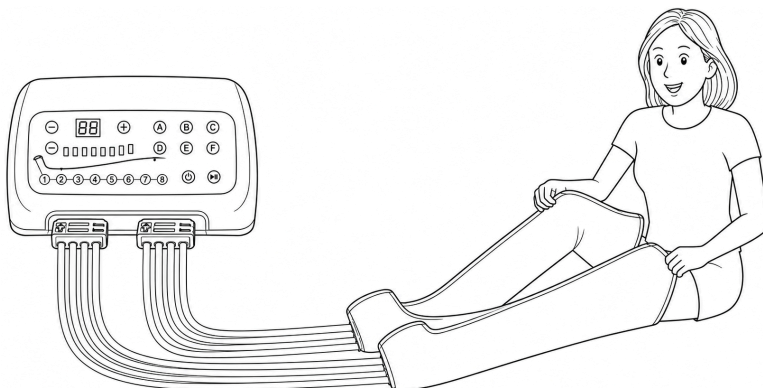
Illustration shown: Nymph 8 model.

NOTE:

It is recommended to switch on the device before fastening the cuffs.

11.3. Starting a treatment session

Step 6

Press the ON/OFF button to switch on the device.

Step 7

Set the desired pressure level by pressing the + and – buttons. The available pressure levels are 30, 60, 90, 120, 150, 180, 210, and 240 mmHg. It is recommended to start with a lower pressure setting during the first few sessions and gradually increase the pressure according to personal preference and comfort.

NOTE:

For first-time users, a pressure setting of 120 mmHg or 150 mmHg is recommended. Increase the pressure gradually if comfortable.

Step 8

Set the treatment time using the + and - buttons. The available range is 1–90 minutes. The default setting is 30 minutes. Press briefly to adjust the time in 1-minute increments or press and hold for rapid adjustment. The device switches off automatically when the timer reaches zero.

11.4. Selecting a program

Step 9

Select the desired program by pressing A, B, C, D, E or F.
(refer to the chapter “MASSAGE PROGRAMS A–F”)

11.5. Enabling and Disabling Chambers

Step 10

Enable or disable individual chambers by pressing the corresponding chamber number. By default, all chambers are enabled, and the indicator lights are on. Press a chamber number once to disable that chamber and switch off its indicator light. Press it again to re-enable the chamber and switch the indicator light back on. The indicator light of a chamber flashes while that chamber is inflating or deflating.

11.6. Starting the program

Step 11

Press the Start/Pause button to begin the treatment session. By default, the device is in pause mode, and the indicator light is green. Press the Start/Pause button to start the treatment. The indicator light changes to blue while the treatment is running.

NOTE:

At the start of treatment, the inflation time may be longer as the cuffs must be filled from a completely empty state.

CAUTION:

- Do not remove the cuffs while the device is operating. Always switch off the device before disconnecting or removing the cuffs.

- Stop treatment and consult your physician if pain, bruising, or irritation occurs during or after treatment.

Recommended session duration

For first-time users, it is recommended to start with a short session of approximately 10 to 20 minutes.

If comfortable, the session duration may be gradually increased during future uses according to personal preference and comfort.

Discontinue use immediately if discomfort, pain, dizziness, numbness, or any unusual reaction occurs.

11.7. After Use

The treatment session ends automatically when the timer reaches zero. You may also stop the session at any time by pressing the Start/Pause button or switching off the device.

CAUTION:

Always switch off the device before removing the cuffs or disconnecting any hoses.

After the device has been switched off, open and remove the cuffs. If required, gently press the air chambers by hand to release any remaining air and speed up deflation.

To disconnect the air hose assembly from the control unit, press the two release clips located on both sides of the main connector. While holding the clips, pull the connector straight out of the control unit. The cuffs and air hose should normally be stored together as one assembly. Avoid disconnecting the hose from the cuffs unnecessarily, as repeated disconnection may damage the hose connectors or pins.


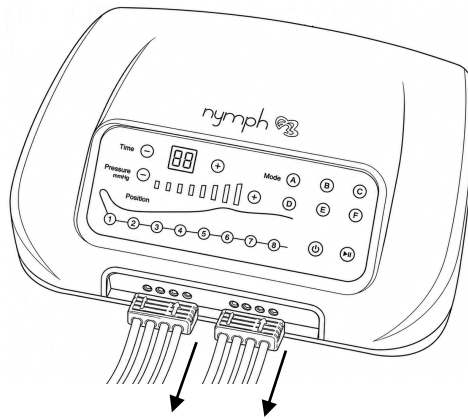
Illustration shown: Nymph 8 model.

NOTE:

For transportation or storage, the air hose may be disconnected from the cuffs to reduce the required storage space.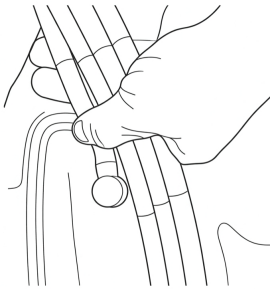

Step 1

Do not pull on the hose.

Step 2

Hold the connector pins and pull them straight out of the cuff connector.

12. USING THE REMOTE CONTROL

- First-time use: If the remote control contains a battery insulation strip, remove it before first use. If the battery is not installed (for example, due to transport regulations), insert a CR2032 battery into the battery compartment.
- Aim the remote control at the IR receiver located on the operating panel of the Nymph 4-6-8. For optimum performance, ensure that the remote control is aligned with the IR receiver. Use the buttons of the remote control to control the Nymph 4-6-8.
- The maximum range of the remote control unit is approximately 5 meters.
- If the remote control does not work properly, take the following steps:
 - Remove all obstacles between the remote control and the Nymph 4-6-8.
 - Reduce the distance between the remote control and the Nymph 4-6-8.
 - Replace the battery with a new one.
 - Contact us if the above does not solve the problem.

NOTES:

- Please remove the protective film from the remote control before use.
- The remote control provides access to all functions available on the Nymph 4-6-8 operating panel.

13. USING THE CUFFS

Preparing for use:

- For comfort and hygiene, wear light clothing underneath the compression garments. Clothing should be clean, comfortable, and free from buttons, zippers, or other hard objects that could cause discomfort during use.
- Remove watches, bracelets, rings, or other jewellery before using arm or leg garments.

Recommended positions:

- Place the control unit on a stable, flat surface where it can be easily reached during operation. Ensure that the air hoses are not twisted, kinked, or obstructed.
- Leg cuffs may be used while sitting or lying down. For increased comfort while sitting, the legs may be slightly elevated.
- Pants cuffs should be used while lying down on a bed, sofa, or similar flat surface. Using pants cuffs while sitting may cause folds in the cuffs that can affect inflation performance.
- Arm garments may be used while sitting or lying down with the arm supported in a comfortable position.
- During use, remain relaxed and avoid excessive movement that may interfere with the inflation and deflation cycle of the compression garments.

WARNING:

Do not walk, run, climb stairs, drive, or perform other activities while wearing leg cuffs or pants cuffs during operation. The cuffs restrict natural movement and may increase the risk of tripping, falling, loss of balance, or injury. Always remain seated or lying down while the cuffs are inflated and in use.

CAUTION:

Do not inflate a cuff unless it is properly worn on the intended body area. Do not operate the system with a cuff that is not being worn or with zippers left open. Doing so may affect performance and may damage the compression garment.

14. TROUBLESHOOTING

① The device does not turn on

1. Make sure the power cord is connected securely to the control unit and plugged into a suitable power outlet.
2. If the device still does not turn on, disconnect it from the power supply. Turn the device over and remove the fuse holder marked "FUSE" using a screwdriver. Check whether the fuse is intact.
3. If the fuse is intact, reinstall the fuse holder and contact customer service for assistance. If the fuse is defective, replace it with a slow-blow fuse (T3.15A, 5 × 20 mm).

② The cuffs inflate in the wrong order

1. Make sure the multi-port connector is inserted into the control unit in the correct orientation. The connector is installed correctly when the locking tabs align with the openings located above the air outlets.

③ The control unit is unusually loud

1. Make sure the transportation screw has been removed before operating the device.
2. Turn the device over and open the filter cover on the bottom of the unit.
3. Remove the transportation screw and dispose of it.
4. Restart the device and check whether the noise level has returned to normal.

15. CLEANING

- A clean, soft, dry cloth is recommended for cleaning the exterior of the control unit.
- Do not wash the cuffs. Wipe gently with a dry cloth.
- Be careful not to put too much pressure on the display and buttons while cleaning.
- Do not use water for cleaning.
- Do not submerge the Nymph 4-6-8 in water or any other liquids. It is not allowed to rinse the control unit with water to prevent water from leaking into the electrical circuit and causing malfunctions.
- Under no circumstances should you use aggressive cleaning agents, cleaning alcohol, or other chemical solutions, as these can penetrate the housing and impair functionality.
- Do not use abrasive objects for cleaning.
- Cleaning must not be performed by children.

16. MAINTENANCE & SERVICE

- The Nymph 4-6-8 is maintenance-free, except for cleaning.
- You may not modify, self-repair, or disassemble the Nymph 4-6-8.
- Use only original parts or parts recommended by the manufacturer.
- Check regularly for damage and severe wear.

WARNINGS:

- Do not attempt to open the Nymph 4-6-8 for maintenance. Removing the housing or improper assembly may cause electric shocks, damage, short circuits, or fire.
- If, for any reason, you open the housing, the warranty will become null and void. The Nymph 4-6-8 contains no user-serviceable parts. Only the battery from the remote control can be replaced.
- Maintenance, service, and repairs may only be carried out by the manufacturer, a professional workshop appointed by the manufacturer, or a similarly qualified person. For maintenance or service, please contact us first at info@nymph.be.

- For missing accessories such as the user manual or other items, contact us at the above-mentioned email address.
- In case of a defect, return the product in the original packaging to **NYMPH** with a failure description, proof of purchase, and all accessories.

17. STORAGE

- We advise cleaning the Nymph 4-6-8 before long-term storage (see the chapter "CLEANING").
- Store the Nymph 4-6-8 preferably in its original packaging.
- Do not store wet parts, as mould or rust may develop.
- Store the Nymph 4-6-8 in a place not exposed to sunlight or humidity, and which is dust-free.
- Do not store the Nymph 4-6-8 in extremely high or extremely low temperatures. This can damage the device.
- Store the Nymph 4-6-8 beyond the reach of young children.
- Remove the battery from the remote control before long-term storage.
- Loosely coil the power cable but do not wrap it around the device.
- Fold the cuffs neatly and do not place heavy objects on top of them, as this may damage the cuffs or air chambers.
- For storage, place the cuffs (incl. tubes) on top of each other. Then roll them loosely together. For transportation or storage, the air hose may be disconnected from the cuffs to reduce the required storage space.

18. TRANSPORT

- Reinstall the transportation screw. The transportation screw is intended for transport only and must not be reinstalled during normal use.
- During transport:
 - Avoid unnecessary shocks and impacts.
 - Do not drop the Nymph 4-6-8.
 - Do not place heavy objects on top of it.
 - Protect the Nymph 4-6-8 from vibrations.
 - Protect the cuffs from puncture damage.
 - Use the original packaging whenever possible.
- After transport or storage at very cold temperatures, the Nymph 4-6-8 must be accustomed to room temperature for approximately 30 minutes due to possible condensation. After that, the device can be used.

19. DISPOSAL & RECYCLING

This product has been manufactured using high-quality materials and components that can be reused and recycled. Please dispose of the product responsibly at the end of its service life.

19.1. Disposal of electrical and electronic equipment (WEEE)

Products marked with the crossed-out wheeled bin symbol are subject to separate collection in accordance with Directive 2012/19/EU (WEEE) and applicable national legislation.

Electrical and electronic equipment must not be disposed of with unsorted household waste. Instead, take the device to:

- an authorised municipal collection point, or
- a retailer upon the purchase of an equivalent new product.

Correct separate collection helps prevent environmental harm and enables proper recycling of valuable materials.

NOTE:

Improper disposal may lead to penalties under national legislation.

19.2. France – Triman marking

If this product is sold in France, disposal instructions apply in accordance with the Triman legislation. Follow local collection guidance as indicated by the Triman logo and official French disposal instructions.

All packaging materials are recyclable. They must be sorted and disposed of in the appropriate recycling streams, in accordance with applicable French regulations and local collection instructions.

19.3. Germany (ElektroG §18(4))

For products sold in Germany, the following additional consumer information is required:

- Old electrical and electronic devices must be collected separately and not disposed of in household waste.
- Batteries and lamps that can be removed without damage must be taken out before disposal.
- Consumers may return old devices free of charge to municipal collection points.
- Retailer take-back obligations in Germany:
 - 1:1 take-back: When purchasing a new device of the same type, the old device must be accepted free of charge.

- 0:1 take-back: Small devices (external dimension < 25 cm) can be returned even without buying a new product, up to three units per type.
- Applies to physical retailers with $\geq 400 \text{ m}^2$ sales area for electrical/electronic goods, and to distance sellers whose storage/shipping areas for such goods cover $\geq 400 \text{ m}^2$.
- Consumers are responsible for deleting all personal data from devices prior to return.
- Germany reports annual collection and recycling quotas under the WEEE Directive.

19.4. Disposal of the batteries

The handling and disposal of batteries and rechargeable batteries are governed by Regulation (EU) 2023/1542 on batteries and waste batteries. This regulation is directly applicable in all EU member states and aims to ensure the safe, sustainable, and environmentally responsible use of batteries.

Consumers are legally obliged not to dispose of batteries in household waste. Instead, used batteries must be returned to designated collection points, as indicated by the crossed-out wheeled bin symbol, for proper recycling. Correct disposal helps to protect the environment and human health.

19.5. Disposal of packing material

Dispose of packaging materials in an environmentally sound manner. Follow the notes and recycling symbols on the packaging and ensure plastic bags, cardboard, and other materials are collected for recycling.

20. WARRANTY & LIABILITY

NYMPH guarantees that for two years from the date of purchase, this product will be free from defects in materials and workmanship. This warranty does not affect the consumer's statutory rights under applicable EU consumer protection laws.

The warranty applies to products used for personal purposes and not for commercial or rental purposes. **NYMPH**, at its discretion, will replace or repair this product or any component of the product found to be defective during the warranty period. Replacement will be made with a new or refurbished product or component. If the product is no longer available, a replacement may be made with a similar product of equal or greater value. No refunds shall be given. Your replacement unit will be returned to you at no charge and will be covered under this warranty for the remainder of the warranty period.

This warranty is valid for the original purchaser from the date of initial purchase and is not transferable. Proof of purchase is required to obtain support under the warranty.

This warranty does not cover:

- Negligent use or misuse of the product.
- Using with incorrect voltage or current.
- Damage resulting from falls or impacts.
- Water damage.
- Use contrary to the operating instructions.
- Disassembly, repair, or modification by anyone other than **NYMPH**.
- Damage caused by accidents or disasters, such as fire, flood, earthquake, war, vandalism, or theft.
- Damage caused by incorrect maintenance and storage.
- Damage caused by incorrect assembly.
- Damage caused by incorrect connections.
- Accessories such as batteries and fuses.
- Exposure to adverse external influences, such as sunlight and other radiation or electromagnetic fields.
- Normal wear and tear.

Damage caused by failure to follow this manual's instructions will void the warranty. We do not accept any liability for consequential damage. We accept no liability for material damage or personal injury caused by improper use or failure to observe the safety instructions. In such cases, the warranty becomes void.

NYMPH reserves the right to make changes at any time, without notice or obligation, in the user manual, colours, specifications, accessories, materials, and models.

All rights reserved. No part of this publication may be reproduced, distributed, or transmitted in any form or by any means, including photocopying, recording, or other electronic or mechanical methods, without the prior written permission of the publisher, except in the case of brief quotations embodied in critical reviews and certain other non-commercial uses permitted by copyright law. For permission requests, contact us.

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