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Air Pressure Massage System

Model Nymph 8 PLUS

This user manual accompanies the Nymph 8 PLUS and contains important information about installation, safety, disposal and use. Read this manual carefully and follow the instructions to ensure safe and proper use of the product.

Keep the manual for further reference, even if you are familiar with the product. Make sure to include this user manual when passing on this product to third parties.

The images in the user manual are for illustrative purposes only and may differ slightly from the product. In that case, **NYMPH** is not obliged to update or revise the user manual. Use the actual product as a reference.

IMPORTANT – READ BEFORE USE

This device is a Class IIa medical device intended for intermittent pneumatic compression therapy.

Use this device only in accordance with its intended use and observe all contraindications, warnings and precautions described in this user manual.

NYMPH accepts no liability for damage, injury or loss resulting from use contrary to this user manual, failure to observe contraindications, warnings or precautions, or use outside the intended purpose.

1. INTRODUCTION

Thank you for choosing a **NYMPH** product. This manual provides the information required for safe and proper operation of the product.

This product bears the CE marking and complies with applicable European regulatory requirements.

	CE marking This product bears the CE marking, indicating that it conforms to the relevant European Union directives concerning safety, health, and environmental protection. The CE marking is a declaration by the manufacturer that the product meets the applicable EU legal requirements.
	RoHS Compliance This product complies with the RoHS Directive (2011/65/EU), which restricts the use of certain hazardous substances in electrical and electronic equipment. Compliance ensures that levels of substances such as lead, mercury, and cadmium remain below the maximum permitted thresholds.
EMC	EMC Compliance This product complies with the Electromagnetic Compatibility (EMC) Directive (2014/30/EU), which ensures that the device functions properly in its environment without causing or suffering from electromagnetic interference.
MDR	MDR Compliance This product complies with the Medical Device Regulation (EU) 2017/745 (MDR), which establishes requirements for the safety, performance, and clinical evaluation of medical devices placed on the European market. Compliance demonstrates that the product meets the essential requirements for medical devices under EU law.

The full EU Declaration of Conformity and all supporting test documentation are available upon request.

If you have any questions, remarks, or problems using the product, please contact us.

We hope you enjoy your purchase!

2. EXPLANATION OF SYMBOLS

Throughout this user manual, the following symbols are used:

	NOTE: Provides important information or useful tips for the use of the product.
	DANGER: Indicates a hazardous situation which, if not avoided, will result in death or serious injury.
	WARNING: Indicates a hazardous situation which, if not avoided, could result in death or serious injury.
	CAUTION: Indicates a hazardous situation which, if not avoided, could result in minor injury or material damage.

The safety symbols do not replace the written safety instructions. Read and follow the safety instructions exactly.

Medical device symbols:

Symbol	Symbol title	Explanation
	Serial number (SN)	Indicates the manufacturer's serial number for product identification.
	Batch code (LOT)	Indicates the manufacturer's batch or lot number.

	Authorised representative in the European Union (EU REP)	Indicates the authorised representative within the European Union.
	Date of manufacture	Indicates the date when the medical device was manufactured
	CE marking with notified body number	Indicates conformity with applicable EU legislation. The number identifies the notified body.
	Type BF applied part	Indicates a Type BF applied part providing protection against electric shock.
	WEEE symbol	Indicates that the device must not be disposed of with household waste. Dispose of the product in accordance with local regulations for waste electrical and electronic equipment (WEEE).
	Indoor use only	Indicates that the device is intended for indoor use only.
	Class II equipment	Indicates Class II equipment with double or reinforced insulation for protection against electric shock.
	IP21 protection rating	Indicates protection against solid foreign objects larger than 12,5 mm and protection against vertically falling water droplets.
	Manufacturer	Indicates the legal manufacturer of the product.

3. TERMINOLOGY

NOTE:

In this user manual, the term “Nymph 8 PLUS” refers to the complete product, including the control unit (device) and the supplied accessories such as the charger and cuff with connecting hose. When individual components are described, they are referred to by their specific names.

4. SAFETY INSTRUCTIONS

Follow the instructions and safety information in this user manual carefully. Failure to do so may result in personal injury or damage to property.

Damage or injury resulting from improper use or failure to follow these instructions is not covered by the warranty.

Electrical devices and batteries may present hazards if used incorrectly. These may include injury, fire, electric shock, or damage to property. Always handle the device with care and keep it out of the reach of children.

This user manual provides the information required for safe use of the device. However, not all possible situations or hazards can be covered. Always use the device with caution and follow the instructions provided.

Residual risks may remain despite all safety measures. Follow all instructions and warnings in this user manual to reduce the risk of injury, damage, or malfunction.

If you have any questions that are not addressed in this user manual, please contact us.

4.1. Contraindications

Do not use the device:

- on open wounds, fractures, or severely injured body areas;
- on areas affected by acute inflammation or infection;
- if you have acute venous thrombosis or deep thrombophlebitis;
- if you suffer from pulmonary edema;
- if you have severe peripheral arterial occlusive disease (PAOD).

Consult a physician before use if you:

- are pregnant;
- have a pacemaker or other implanted electronic device;
- suffer from heart disease, circulatory disorders, or uncontrolled hypertension;
- have diabetes with reduced sensation or peripheral neuropathy;
- are undergoing cancer treatment;
- take anticoagulant medication;
- have any other medical condition that may be affected by compression massage.

4.2. General safety

- Do not use the Nymph 8 PLUS for any purpose other than its intended use (see the chapter "INTENDED USE").
- The Nymph 8 PLUS is designed and intended for household use and is not suitable for commercial or industrial use.
- Do not use the device in places where the use of electronic equipment is not allowed.
- The Nymph 8 PLUS is intended for indoor use only. Do not use the device outdoors.
- Handle the device carefully. Shocks, impacts, or falls, even from a low height, can damage the device.
- Do not place the Nymph 8 PLUS on an unstable surface. This may cause it to fall and result in a malfunction.
- Do not squeeze the cuff.
- Do not expose the Nymph 8 PLUS to excessive vibration, crushing forces, impacts, or other mechanical stress.
- Do not sit on the cuff.

- The device must be properly and securely connected. Follow the instructions to prevent any possible damage or injury.
- Use only the accessories or attachments supplied or recommended by the manufacturer for this device. Incorrect accessories or attachments may pose a risk to the user and can damage the Nymph 8 PLUS.
- Do not insert any objects or fingers into openings.
- Do not apply excessive force when using the buttons on the Nymph 8 PLUS.
- Do not touch the cuff with sharp objects.
- Do not apply excessive force to the display, and do not touch the display with sharp objects.
- Never place objects on the Nymph 8 PLUS.
- Do not leave the switched-on Nymph 8 PLUS unattended. Close supervision is necessary when using near young children.
- Do not use the Nymph 8 PLUS near strong magnetic fields, e.g. motors, transformers, mobile phones, etc.
- Do not use the Nymph 8 PLUS in dusty environments.
- If it is no longer possible to operate the Nymph 8 PLUS safely, take it out of operation and protect it from any accidental use. A safe operation can no longer be guaranteed if the device:
 - is visibly damaged;
 - is no longer working properly;
 - has been stored for extended periods in poor ambient conditions;
 - or has been subjected to serious transport-related stresses.

4.3. Users

- The Nymph 8 PLUS is not a toy. Keep it out of reach of children and pets.
- The device may only be used by adults. The cuffs are designed for adult users only. Using them on children may result in injury.
- Do not allow persons who are not familiar with the operation of the Nymph 8 PLUS or with these instructions to use the device.
- The device may only be used by persons who understand these instructions and are aware of the potential hazards.
- This device is not intended for use by persons with reduced physical, sensory, or mental capabilities or a lack of experience and knowledge, unless they have been given supervision or instruction concerning the use of the device by a person responsible for their safety.
- Cleaning and maintenance must not be performed by children.

- Do not operate the Nymph 8 PLUS when you are tired, distracted, or under the influence of alcohol or medication.

4.4. Electrical safety & charger

- You must not modify, repair yourself, or disassemble the Nymph 8 PLUS. This can damage the device.
- Maintenance, modifications, and repairs may only be carried out by the manufacturer, a professional workshop appointed by the manufacturer, or a similarly qualified person.
- Check the device from time to time for damage. Never use the device if it shows signs of damage.
- Do not use the device while charging the battery.
- Only use the supplied charger to charge the Nymph 8 PLUS. Another charger with the wrong specifications may cause malfunction and/or damage.
- To prevent damage to the charger and/or the Nymph 8 PLUS, ensure your mains outlet supplies the correct power supply for the charger.
- The mains outlet must be located near the Nymph 8 PLUS and be easily accessible.
- To reduce energy consumption and potential safety risks, always unplug the device when it is not being charged, during cleaning or maintenance, or if a fault occurs. The device remains connected to the mains as long as the charger is plugged in.
- Never force the plug of the charger into a mains outlet. If it does not fit easily into the mains outlet, stop using it. Always ensure that the plug of the charger is properly connected to the mains outlet.
- It is normal for the charger to feel a little warm. However, a loose fit between the mains outlet and the plug of the charger can cause overheating and deformation of the plug. Contact a qualified electrician to replace the loose or worn mains outlet.
- To disconnect the charger, grasp the charger housing and pull the charger out of the mains outlet.
- If the power fails during charging, pull the charger immediately from the mains outlet.
- If smoke is produced, or if there are any unusual sounds or smells during operation of the Nymph 8 PLUS or while charging the battery, switch off the device immediately and disconnect the charger from the mains outlet. In such cases, the Nymph 8 PLUS and the battery must not be used until they have

been inspected by the manufacturer or a similarly qualified person. Never inhale smoke from a potential device or battery fire. If smoke has been inhaled, seek medical advice, as inhalation of smoke can be harmful to health.

- In the event of a fire, disconnect the power supply if this can be done safely. Use suitable fire-fighting equipment and keep a safe distance from the device.
- For safety reasons, always disconnect the charger in the event of a thunderstorm.

4.5. Charger cable

- Fully unwind the cable before use.
- Do not modify the charger cable.
- Do not use the charger if the cable is damaged. Replace the charger with a damaged cable immediately with a suitable replacement.
- To prevent risks, damaged cables may only be replaced by the manufacturer, a professional workshop appointed by the manufacturer, or a similarly qualified person.
- Never force the DC connector into the connection port. If the connector and port do not fit together easily, they are probably not compatible. Check for obstructions and ensure the connector matches the port, and that you have correctly positioned the connector with the port.
- Never unplug the charger cable from the Nymph 8 PLUS by pulling the cable. Always pull on the DC-connector at the end of the cable.
- Do not carry or pull the Nymph 8 PLUS by its charger cable.
- Never wrap the charger cable around the device. This will cause the cable to wear prematurely and break.
- Be careful not to trip over the cable.
- The cable must not be kinked, pinched, or twisted.
- Ensure that the cable is not squeezed, bent, damaged by sharp edges, or put under mechanical stress.
- Never tie a knot in the cable.
- Untwist the cable periodically if it becomes twisted due to use.
- Avoid excessive thermal stress on the cable due to extreme heat or cold.
- Children can become entangled while playing with cables, posing a strangulation hazard. Electric cables are not toys.

4.6. Heat & ventilation

- The Nymph 8 PLUS may become warm during operation. This is normal.
- Do not cover the device or the ventilation holes while it is in operation, as this may cause the Nymph 8 PLUS to overheat. This poses a risk of fire, electric shock, or injury.
- Ensure that the ventilation around the Nymph 8 PLUS is good during use.
- Keep the device away from heat sources and away from areas where the ambient temperature is relatively high (e.g. stoves or other heat sources). Never use the device near naked flames (such as burning candles) or on hot surfaces.
- The Nymph 8 PLUS is designed to work best in normal ambient temperatures between 0 °C and 35 °C and to be stored at temperatures between 0 °C and 45 °C. The Nymph 8 PLUS may be damaged, and the lifespan may be shortened if the device is used or stored outside this temperature range. Do not subject your device to dramatic changes in temperature. Do not store your device in a hot car or direct sunlight for extended periods. Do not place the device near a window (especially in the summer).

4.7. Water

- This device is not waterproof. Exposure to water may result in electric shock, short circuits, or permanent damage.
- Never use the Nymph 8 PLUS with moist hands. Moisture can cause electric shock or short circuits.
- Never touch the plug with moist hands while connecting or disconnecting it from the mains outlet. Moisture can cause electric shock or short circuits.
- Do not use the Nymph 8 PLUS in or near a bath, shower, swimming pool, or any container filled with water, such as a sink.
- Do not use in wet environments. Do not immerse the device, charger or cuff in any liquid. Never pour liquids over or near the Nymph 8 PLUS. There is a risk of electric shock or a short circuit. If liquid does get into the housing, unplug it immediately and then turn off the Nymph 8 PLUS.
- Do not reach for the device if it has fallen into the water. Immediately unplug it from the mains outlet and only then remove it from the water.
- Ensure that the Nymph 8 PLUS is placed on a dry surface.

4.8. Battery safety

- Do not place Nymph 8 PLUS near a fire or in direct sunlight. The heating of the device can cause additional heating, breaking, or explosion of the battery.
- Do not charge the battery at temperatures below 5°C or above 40°C.
- Charge the battery in a well-ventilated area.
- Do not continue charging the battery if it does not recharge within the specified charging time. Doing so may cause the battery to become hot, rupture, or ignite. Unplug the device once fully charged or if left unattended during charging.
- Do not attempt to charge the battery if the battery emits any substances. In that case, immediately distance yourself from the battery in case of fire or explosion. Only charge with the supplied charger.
- Do not charge the battery near flammable materials (fire hazard).
- Do not attempt to pierce, modify, disassemble, or change the battery, as this may result in fire, explosion, chemical leakage, or serious injury.
- Do not use a battery that is damaged or deformed.
- Do not use the Nymph 8 PLUS if the battery begins to emit odour, starts to overheat, or begins to leak.
- Do not touch any leaking materials or breathe fumes emitted from the battery. Leaking or damaged batteries can lead to caustic burning of the skin. Therefore, use suitable gloves.
- The battery contains hazardous substances. Do not open the battery or insert any objects into it.
- Never throw the battery into a fire.
- Even a fully discharged battery may still contain hazardous residual energy and must be handled carefully.
- When handling the battery, ensure your hands, the product, and the battery are dry.
- Do not short-circuit. Short-circuiting may cause the battery to explode or leak.
- To avoid accidental short-circuiting of the battery after removal, do not let the battery terminals come into contact with metal objects (e.g. coins, hairpins, rings). Do not wrap the battery in aluminium foil. Tape the battery terminals or put the battery in a plastic bag before you discard it.
- Prevent children or animals from coming into contact with the battery.

4.9. Residual Risks

Despite the implementation of all safety measures, certain residual risks may remain when using the Nymph 8 PLUS.

- Temporary discomfort, skin irritation, redness, numbness, or pressure marks caused by prolonged use or excessive pressure settings.
- Discomfort or injury resulting from improper placement or incorrect fitting of the cuffs.
- Reduced blood circulation, pain, or physical discomfort caused by excessive treatment duration or unsuitable pressure settings.
- Electric shock, overheating, or fire hazards resulting from damaged cables, damaged components, improper repairs, exposure to liquids, or misuse of the device.
- Tripping hazards caused by hoses, cables, or connected garments during use or storage.
- Damage to the cuffs, hoses, connectors, or control unit caused by sharp objects, excessive bending, pulling, twisting, crushing, or improper storage.
- Risks associated with use by persons with contraindicated medical conditions, including circulatory disorders, thrombosis, severe hypertension, implanted medical devices, or other conditions listed in this manual.
- Risks associated with use by children, persons with reduced physical, sensory, or mental capabilities, or persons unable to understand the instructions or communicate discomfort.

To minimise these risks, always follow all instructions and safety warnings provided in this manual.

4.10. Reasonably foreseeable misuse

The following uses are considered reasonably foreseeable misuse and must be avoided:

- Using the device for purposes other than intermittent pneumatic compression therapy using the supplied compression garments.
- Using the device on infants, children, pets, or persons unable to safely operate the device or communicate discomfort.
- Using the device while sleeping, unconscious, under the influence of alcohol, medication, or substances that may reduce awareness or sensitivity.
- Using the device with damaged cuffs, hoses, connectors, cables, plugs, or other components.
- Using the device in wet environments, near water, or with wet hands.
- Covering the control unit during operation or blocking the ventilation openings.
- Pulling, twisting, sharply bending, crushing, or stepping on hoses or cables.
- Disconnecting the cuffs or hoses while the device is operating or pressurised.
- Using pressure settings, treatment durations, or programs that cause pain, numbness, or significant discomfort.
- Using the device despite known contraindications, medical conditions, injuries, or after medical advice not to use compression therapy devices.
- Attempting to open, modify, repair, or disassemble the device or accessories.
- Inserting foreign objects into the control unit, connectors, ventilation openings, or other openings of the device.
- Using non-approved accessories, replacement parts, chargers or cuffs.
- Standing, walking, or moving around while connected to the compression garments.
- Using the product outdoors or exposing it to direct sunlight, excessive heat, open flames, or freezing temperatures.

Such misuse may result in injury, fire hazards or damage to the device.

4.11. Medical considerations

This product is a Class IIa medical device intended for intermittent pneumatic compression therapy. It should only be used in accordance with its intended purpose and should not replace professional medical advice, diagnosis, monitoring, or treatment.

Consult a physician before using this device if you:

- suffer from circulatory disorders, vascular disease, varicose veins, diabetes, severe hypertension, or heart disease;
- have implanted medical devices such as a pacemaker;
- are pregnant or have recently undergone surgery;
- experience chronic pain, numbness, reduced sensitivity, or skin conditions;
- are undergoing medical treatment or have any other medical concerns.

Do not use the device on injured, swollen, infected, inflamed, or sensitive body areas unless approved by a qualified healthcare professional.

Stop using the device immediately and seek medical advice if you experience pain, dizziness, numbness, skin discolouration, unusual swelling, breathing difficulties, or any other abnormal physical reaction during or after use.

Users are responsible for determining whether this product is suitable for their individual physical condition.

5. INTENDED USE

The Nymph 8 PLUS is intended for intermittent pneumatic compression therapy using inflatable compression garments for the legs, arms, waist or lower body. The device provides sequential air pressure compression.

The device is equipped with a rechargeable battery, allowing cordless operation during use.

The device is intended to support blood circulation and temporarily relieve minor muscle discomfort and fatigue.

**WARNING:**

Use this product only as described in this manual. Use outside the intended purpose or contrary to the instructions may result in reduced performance, damage to the device, or safety risks.

6. SCOPE OF DELIVERY

6.1. Contents

The following components are included in the scope of delivery:

- 1x Nymph 8 PLUS
- 1x Air hose set
- 1x Charger
- 1x Cuff
- 1x Carrying bag
- 1x User manual
- 1x Shoulder strap for carrying bag

**NOTE:**

Due to transport and packaging requirements, some accessories may be supplied disassembled. Follow the assembly instructions carefully.

6.2. Unpacking

Remove everything from the packaging and check for damage. If any damage is detected, do not use the Nymph 8 PLUS. Be very careful when opening the package to avoid damage to the product.

After unpacking the product, verify that the delivery is complete and that all parts are in good condition. If a part is missing or damaged, please contact us.

Before use, remove all packaging materials.

NOTE:

This device is packaged to protect it during transport. Keep the original box and other packaging in a safe place to prevent damage to the device if you need to transport it in the future. If you do throw away the packaging, please ensure it is recycled properly.

6.3. Safety during unpacking

Do not leave packaging materials and accessories lying around carelessly during unpacking. They may become a dangerous plaything for children.

WARNING: Suffocation hazard

Keep plastic bags and foil out of the reach of babies and children to prevent suffocation. Thin foils can stick to the nose and mouth and obstruct breathing.

**WARNING:** Strangulation hazard

Cables and air hoses longer than 22 cm can be harmful to babies and very young children.

Children may become entangled while playing with the charger cable and hoses. Do not leave the accessories lying around during unpacking.

7. TECHNICAL DATA

**NOTE:**

Due to continuous improvements to our product, the technical specifications may be changed without prior notice.

Product	Air pressure massage system
Model	Nymph 8 PLUS
Brand	Nymph
Colour	Black
Material	Control unit: ABS Cuff: Durable fabric Nylon + TPU
Display	LED
Operation	Touch screen
Number of air chambers	8
Number of programs	6
Power	35 W
Charger input	110 – 240 V AC; 50 – 60 Hz
Charger output	12.9 V DC – 2 A

Protection class (charger)	Class II	
Protection class (device)	Class III	
Battery type	Lithium ion	
Battery capacity	5000 mAh	
Battery voltage	10.8 V DC	
Charging time	5 hours	
Operating time	3 hours	
Charger cable length	1.5 m	
Timer setting	5 - 99 min	
Pressure	30 - 240 mmHg (± 30 mmHg)	
IP rating	IP20	
Weight	1.72 kg	
Dimensions (L x W x H)	245 x 150 x 90 mm	

Product identification:

Brand	Nymph
Model	Nymph 8 PLUS
OEM Manufacturer	Xiamen Weiyou Intelligent Technology Co., Ltd., Xiamen, China
OEM Model	VU-IPCU8

NOTE:

The product identification and rating label are located on the device. This label contains important information such as the model number and electrical ratings.

MDR Compliance:

This product is a Class IIa medical device and complies with the Medical Device Regulation (EU) 2017/745 (MDR), which establishes requirements for the safety, performance, and clinical evaluation of medical devices placed on the European market. Compliance demonstrates that the product meets the essential requirements for medical devices under EU law.


Notified Body:

Notified Body 0598 – SGS Fimko Oy
Takomotie 8, FI-00380 Helsinki, Finland
Tel.: +358 9 696 361
www.sgs.fi
Business ID 0634247-4

Authorised representative in the European Union (EU REP):

SUNGO Cert GmbH
Harffstr. 47, 40591 Düsseldorf, Germany

8. PRODUCT OVERVIEW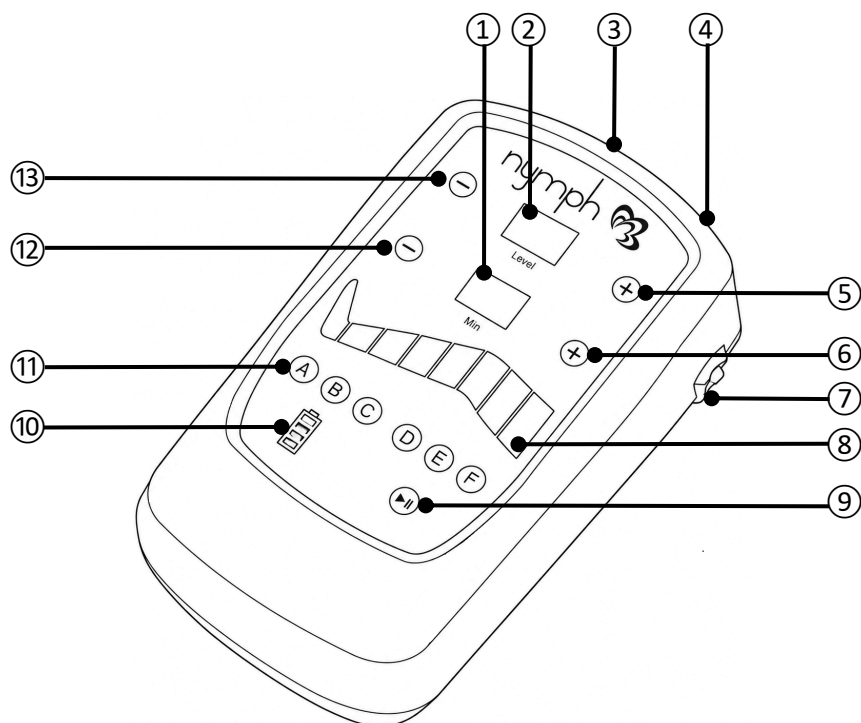

- | | |
|-------------------------------|-------------------------------|
| ① Time display | ⑧ Position/Chamber indicators |
| ② Pressure display | ⑨ Start/Pause button |
| ③ Multi-port connector socket | ⑩ Battery indicator |
| ④ Charging input port | ⑪ 6 massage programs (A-F) |
| ⑤ Add pressure button | ⑫ Subtract time button |
| ⑥ Add time button | ⑬ Subtract pressure button |
| ⑦ ON/OFF switch | |

9. MESSAGE PROGRAMS A–F

The air chambers inside the leg cuffs inflate and deflate according to the selected program. This creates a compression massage according to the selected program. The treatment time, pressure level, and additional settings can be adjusted at any time during the session.

A – Sequential Mode

In this mode, only one chamber is inflated at a time. Inflation starts with chamber 1 and progresses sequentially to chamber 8. Once chamber 8 has completed its cycle, the sequence repeats.

B – Progressive Mode

In this mode, chamber 1 inflates and remains inflated. The device then gradually inflates each subsequent chamber until all eight chambers are inflated. Once the cycle is complete, it repeats.

C – Full Compression Mode

In this mode, all chambers inflate and deflate simultaneously. The cycle then repeats.

D – Two-Chamber Wave Mode

In this mode, chamber 1 inflates and maintains pressure. Chamber 2 then inflates while chamber 1 remains inflated. Next, chamber 1 deflates as chamber 3 inflates, so chambers 2 and 3 remain inflated together. This pattern continues, maintaining pressure in two adjacent chambers while progressing sequentially to chamber 8. The cycle then repeats.

E – Three-Chamber Wave Mode


In this mode, chamber 1 inflates and maintains pressure. Chamber 2 then inflates, followed by chamber 3, so that chambers 1, 2, and 3 are inflated simultaneously. Chamber 1 then deflates as chamber 4 inflates, maintaining pressure in chambers 2, 3, and 4. This pattern continues, maintaining pressure in three adjacent chambers while progressing sequentially to chamber 8. The cycle then repeats.

F – Pulse Wave Mode

In this mode, each chamber inflates to the selected pressure and then performs four rapid inflation and deflation pulses before maintaining pressure. As each new chamber completes its pulse sequence, the previously inflated chambers remain pressurised. Once three adjacent chambers are pressurised, the compression wave progresses by releasing the oldest chamber while the next chamber inflates, pulses four times, and then maintains pressure. This pattern continues sequentially to chamber 8, after which the cycle repeats.

NOTE:

A pulse consists of four rapid inflation and deflation cycles.

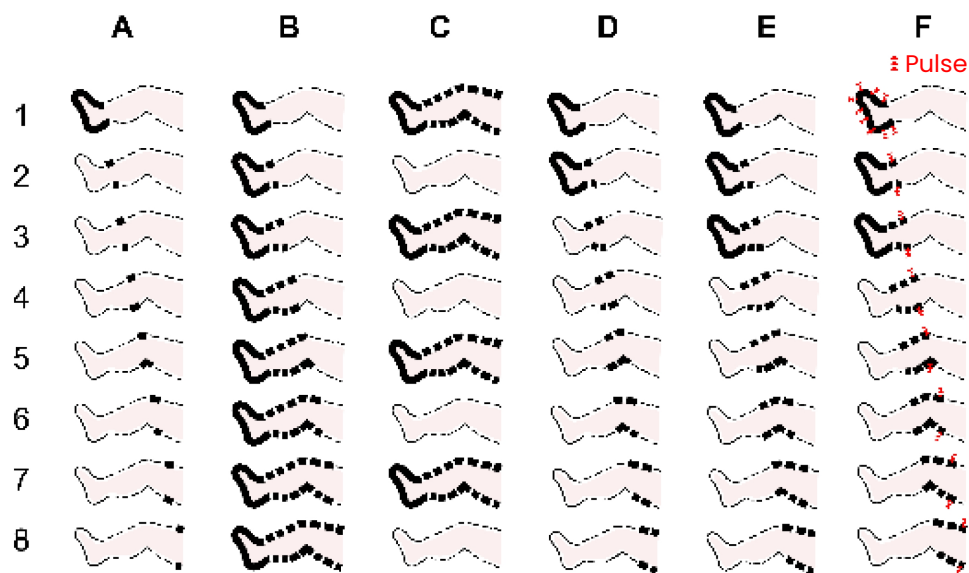10. BENEFITS OF COMPRESSION MASSAGE

The device provides a sequential compression massage using multiple air chambers that inflate and deflate according to the selected program. This creates a gentle pressure massage designed to enhance comfort and relaxation.

Potential benefits of regular use may include:

- Relief from the feeling of tired or heavy legs after prolonged standing, sitting, or daily activities.
- Increased comfort and relaxation after physical activity or exercise.
- A soothing massage experience for the legs or arms.
- Temporary relief from muscle tension and physical fatigue.
- Improved comfort during periods of rest and recovery.


NOTES:

- The benefits described above are intended to support compression therapy, and individual experiences may vary.
- This medical device is intended for intermittent pneumatic compression therapy as described in this manual. It is not intended to replace professional medical advice, diagnosis, or treatment.

11. OPERATION

11.1. Before first use

Check the battery level before use. Press the ON/OFF switch to power on the device. If the device does not start, recharge the battery before use.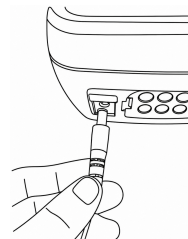

Charging input port



11.2. Use


CAUTION:

Only use the supplied charger or a charger with the specified output rating. Using an incompatible charger may damage the device.

1. Connect the air hoses securely to the compression garments.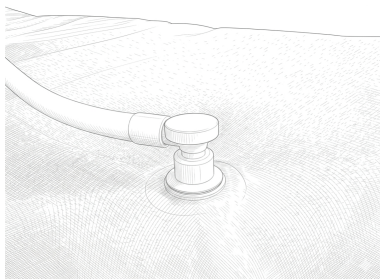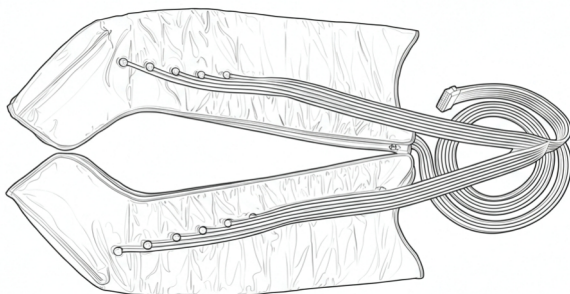

2. Connect the main hose connector securely to the control unit.

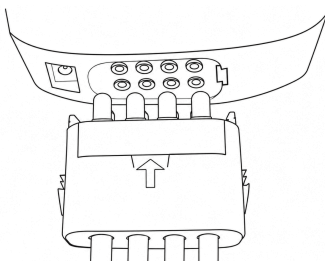Check that all connections are secure and airtight.

3. Press the ON/OFF switch to turn on the device.

NOTE:

The ON/OFF switch is located on the right side of the control unit. Press the switch again to turn off the device.

4. Put on the compression garments comfortably according to the instructions.

NOTE:

It is recommended to switch on the device before fastening the cuffs.

5. Set the pressure level

- Press the + or - button to adjust the desired pressure level.
- The default pressure level is Level 8 (85 mmHg).
- Pressure levels correspond to the following pressure values:
 - Levels 1–4: 30–75 mmHg
 - Levels 5–8: 85–130 mmHg
 - Levels 9–12: 140–185 mmHg
 - Levels 13–16: 195–240 mmHg

NOTE:

For first-time users, a pressure setting of 120 mmHg or 150 mmHg is recommended. Increase the pressure gradually if comfortable.


6. Select the active chambers

- Press the + or - button to set the desired treatment time.
- The treatment time can be set from 5 to 99 minutes.
- The default treatment time is 30 minutes.
- Press the button briefly to increase or decrease the time by 1 minute.
- Press and hold the button to adjust the time more quickly.
- The device switches off automatically when the treatment time has elapsed.

7. Select the treatment area

Press the corresponding chamber button to enable or disable individual chambers.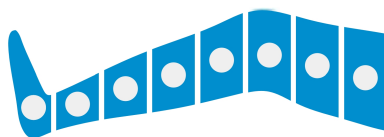

NOTE:

By default, all eight chambers are enabled. Chambers can only be enabled or disabled while the device is in pause mode. Press a chamber button to toggle the corresponding chamber on or off. The indicator changes accordingly.

8. Select the massage mode

Press one of the A–F buttons to select the desired massage mode.

NOTE:

The default massage mode is Mode A.

9. Start the treatment

Press the Start/Pause button to begin the treatment session.

NOTE:

At the start of treatment, the inflation time may be longer as the cuffs must be filled from a completely empty state.

CAUTION:

- Do not remove the cuffs while the device is operating. Always switch off the device before disconnecting or removing the cuffs.
- Stop treatment and consult your physician if pain, bruising, or irritation occurs during or after treatment.

Recommended session duration

For first-time users, it is recommended to start with a short session of approximately 10 to 20 minutes.

If comfortable, the session duration may be gradually increased during future uses according to personal preference and comfort.

Discontinue use immediately if discomfort, pain, dizziness, numbness, or any unusual reaction occurs.

11.3. After Use



The treatment session ends automatically when the timer reaches zero. You may also stop the session at any time by pressing the Start/Pause button or switching off the device.

**CAUTION:**

Always switch off the device before removing the cuffs or disconnecting any hoses.


After the device has been switched off, open and remove the cuffs. If required, gently press the air chambers by hand to release any remaining air and speed up deflation.

To disconnect the air hose assembly from the control unit, press the two release clips located on both sides of the main connector. While holding the clips, pull the connector straight out of the control unit. The cuffs and air hose should normally be stored together as one assembly. Avoid disconnecting the hose from the cuffs unnecessarily, as repeated disconnection may damage the hose connectors or pins.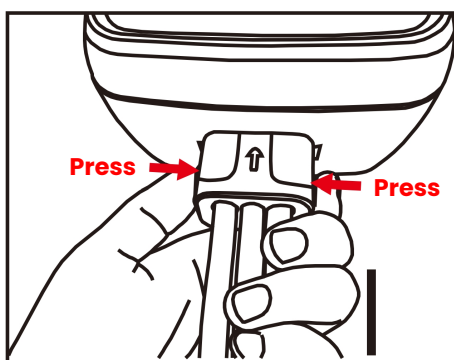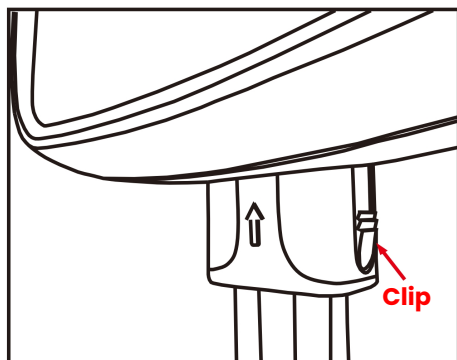

**NOTE:**

For transportation or storage, the air hose may be disconnected from the cuffs to reduce the required storage space.

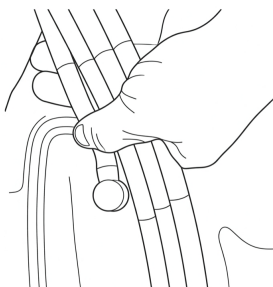**Step 1**

Do not pull on the hose.

**Step 2**

Hold the connector pins and pull them straight out of the cuff connector.

12. USING THE CUFFS

Preparing for use:

- For comfort and hygiene, wear light clothing underneath the compression garments. Clothing should be clean, comfortable, and free from buttons, zippers, or other hard objects that could cause discomfort during use.
- Remove watches, bracelets, rings, or other jewellery before using arm or leg garments.

Recommended positions:

- Place the control unit on a stable, flat surface where it can be easily reached during operation. Ensure that the air hoses are not twisted, kinked, or obstructed.
- Leg cuffs may be used while sitting or lying down. For increased comfort while sitting, the legs may be slightly elevated.
- Pants cuffs should be used while lying down on a bed, sofa, or similar flat surface. Using pants cuffs while sitting may cause folds in the cuffs that can affect inflation performance.
- Arm garments may be used while sitting or lying down with the arm supported in a comfortable position.
- During use, remain relaxed and avoid excessive movement that may interfere with the inflation and deflation cycle of the compression garments.

WARNING:

Do not walk, run, climb stairs, drive, or perform other activities while wearing leg cuffs or pants cuffs during operation. The cuffs restrict natural movement and may increase the risk of tripping, falling, loss of balance, or injury. Always remain seated or lying down while the cuffs are inflated and in use.

CAUTION:

Do not inflate a cuff unless it is properly worn on the intended body area. Do not operate the system with a cuff that is not being worn or with zippers left open. Doing so may affect performance and may damage the compression garment.

13. CLEANING

- A clean, soft, dry cloth is recommended for cleaning the exterior of the control unit.
- Do not wash the cuffs. Wipe gently with a dry cloth.
- Be careful not to put too much pressure on the display and buttons while cleaning.
- Do not use water for cleaning.
- Do not submerge the Nymph 8 PLUS in water or any other liquids. Do not rinse the control unit with water to prevent water from leaking into the electrical circuit and causing malfunctions.
- Under no circumstances should you use aggressive cleaning agents, cleaning alcohol, or other chemical solutions, as these can penetrate the housing and impair functionality.
- Do not use abrasive objects for cleaning.
- Cleaning must not be performed by children.

14. MAINTENANCE & SERVICE

- The Nymph 8 PLUS is maintenance-free, except for cleaning.
- You may not modify, self-repair, or disassemble the Nymph 8 PLUS.
- Use only original parts or parts recommended by the manufacturer.
- Check regularly for damage and severe wear.

WARNINGS:

- Do not attempt to open the Nymph 8 PLUS for maintenance. Removing the housing or improper assembly may cause electric shocks, damage, short circuits, or fire.
- If, for any reason, you open the housing, the warranty will become null and void. The Nymph 8 PLUS contains no user-serviceable parts.
- Maintenance, service, and repairs may only be carried out by the manufacturer, a professional workshop appointed by the manufacturer, or a similarly qualified person. For maintenance or service, please contact us first at info@nymph.be.

- For missing accessories such as the user manual or other items, contact us at the above-mentioned email address.
- In case of a defect, return the product in the original packaging to **NYMPH** with a failure description, proof of purchase, and all accessories.

15. STORAGE

- We advise cleaning the Nymph 8 PLUS before long-term storage (see the chapter "CLEANING").
- Store the Nymph 8 PLUS preferably in its original packaging or use the included carrying bag.
- Do not store wet parts, as mould or rust may develop.
- Store the Nymph 8 PLUS in a place not exposed to sunlight and humidity, and which is dust-free.
- Do not store the Nymph 8 PLUS in extremely high or extremely low temperatures. This can damage the device.
- Store the Nymph 8 PLUS beyond the reach of young children.
- Recharge the battery of the Nymph 8 PLUS every 3 months, even if you did not use it at all.
- Loosely coil the charger cable, but do not wrap it around the device.
- Fold the cuffs neatly and do not place heavy objects on top of them, as this may damage the cuffs or air chambers.
- For storage, place the cuffs (incl. tubes) on top of each other. Then roll them loosely together. For transportation or storage, the air hose may be disconnected from the cuffs to reduce the required storage space.

16. TRANSPORT

- During transport:
 - Avoid unnecessary shocks and impacts.
 - Do not drop the Nymph 8 PLUS.
 - Do not place heavy objects on top of it.
 - Protect the Nymph 8 PLUS from vibrations.
 - Protect the cuffs from puncture damage.
 - Use the original packaging whenever possible or the included carrying bag.
 - Ensure the Nymph 8 PLUS is switched off.
- After transport or storage at very cold temperatures, the Nymph 8 PLUS must be accustomed to room temperature for approximately 30 minutes due to possible condensation. After that, the device can be used.

17. DISPOSAL & RECYCLING

This product has been manufactured using high-quality materials and components that can be reused and recycled. Please dispose of the product responsibly at the end of its service life.

17.1. Disposal of electrical and electronic equipment (WEEE)

Products marked with the crossed-out wheeled bin symbol are subject to separate collection in accordance with Directive 2012/19/EU (WEEE) and applicable national legislation.

Electrical and electronic equipment must not be disposed of with unsorted household waste. Instead, take the device to:

- an authorised municipal collection point, or
- a retailer upon the purchase of an equivalent new product.

Correct separate collection helps prevent environmental harm and enables proper recycling of valuable materials.

NOTE:

Improper disposal may lead to penalties under national legislation.

17.2. France – Triman marking

If this product is sold in France, disposal instructions apply in accordance with the Triman legislation. Follow local collection guidance as indicated by the Triman logo and official French disposal instructions.

All packaging materials are recyclable. They must be sorted and disposed of in the appropriate recycling streams, in accordance with applicable French regulations and local collection instructions.


17.3. Germany (ElektroG §18(4))

For products sold in Germany, the following additional consumer information is required:

- Old electrical and electronic devices must be collected separately and not disposed of in household waste.
- Batteries and lamps that can be removed without damage must be taken out before disposal.
- Consumers may return old devices free of charge to municipal collection points.
- Retailer take-back obligations in Germany:
 - 1:1 take-back: When purchasing a new device of the same type, the old device must be accepted free of charge.

- 0:1 take-back: Small devices (external dimension < 25 cm) can be returned even without buying a new product, up to three units per type.
- Applies to physical retailers with $\geq 400 \text{ m}^2$ sales area for electrical/electronic goods, and to distance sellers whose storage/shipping areas for such goods cover $\geq 400 \text{ m}^2$.
- Consumers are responsible for deleting all personal data from devices prior to return.
- Germany reports annual collection and recycling quotas under the WEEE Directive.

17.4. Disposal of the battery

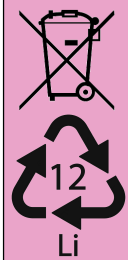This device contains a built-in rechargeable battery. The battery cannot be replaced by the user. The use and disposal of batteries and rechargeable batteries are governed by Regulation (EU) 2023/1542 on batteries and waste batteries. This regulation is directly applicable in all EU member states and aims to ensure the safe, sustainable, and environmentally responsible use of batteries.

Do not dispose of the device and the battery with household waste. Instead, return the device to a designated collection point or to an authorised service centre, as indicated by the crossed-out wheeled bin symbol, so that the battery can be properly recycled. Correct disposal helps protect the environment and human health.

17.5. Disposal of packing material

Dispose of packaging materials in an environmentally sound manner. Follow the notes and recycling symbols on the packaging and ensure plastic bags, cardboard, and other materials are collected for recycling.

18. WARRANTY & LIABILITY

NYMPH guarantees that for two years from the date of purchase, this product will be free from defects in materials and workmanship. This warranty does not affect the consumer's statutory rights under applicable EU consumer protection laws.

The warranty applies to products used for personal purposes and not for commercial or rental purposes. **NYMPH**, at its discretion, will replace or repair this product or any component of the product found to be defective during the warranty period. Replacement will be made with a new or refurbished product or component. If the product is no longer available, a replacement may be made with a similar product of equal or greater value. No refunds shall be given. Your replacement unit will be returned to you at no charge and will be covered under this warranty for the remainder of the warranty period.

This warranty is valid for the original purchaser from the date of initial purchase and is not transferable. Proof of purchase is required to obtain support under the warranty.

This warranty does not cover:

- Negligent use or misuse of the product.
- Using with incorrect voltage or current.
- Damage resulting from falls or impacts.
- Water damage.
- Use contrary to the operating instructions.
- Disassembly, repair, or modification by anyone other than **NYMPH**.
- Damage caused by accidents or disasters, such as fire, flood, earthquake, war, vandalism, or theft.
- Damage caused by incorrect maintenance and storage.
- Damage caused by incorrect assembly.
- Damage caused by incorrect connections.
- Exposure to adverse external influences, such as sunlight and other radiation or electromagnetic fields.
- Normal wear and tear.

Damage caused by failure to follow this manual's instructions will void the warranty. We do not accept any liability for consequential damage. We accept no liability for material damage or personal injury caused by improper use or failure to observe the safety instructions. In such cases, the warranty becomes void.

NYMPH reserves the right to make changes at any time, without notice or obligation, in the user manual, colours, specifications, accessories, materials, and models.

All rights reserved. No part of this publication may be reproduced, distributed, or transmitted in any form or by any means, including photocopying, recording, or other electronic or mechanical methods, without the prior written permission of the publisher, except in the case of brief quotations embodied in critical reviews and certain other non-commercial uses permitted by copyright law. For permission requests, contact us.

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